



# CLEAR CLEAN & POLISH BRITE

## GP LENS CARE CLEANING INSTRUCTIONS

### CONTEX CLEAR CLEAN

- Prevents Protein Deposits & Ensures Optimum Comfort
- Provides Clean Lenses for Great Vision
- Hypoallergenic / Preservative Free Formula
- Natural Formula Cleans & Conditions GP Lenses While they Soak
- Leaves a Comfort Film on Lenses to Ensure Immediate Comfort

Contex's high quality standards and innovative formulas have provided "clean comfortable" lenses for thousands of patients for over 40 years!

**Indications:** For cleaning and conditioning gas permeable contact lenses prior to insertion. After disinfecting lenses, soak them in Clear Clean whenever they are not being worn to condition lenses for optimum comfort. Clear Clean is a surfactant cleanser only and is not a disinfectant. Do not use Clear Clean directly in the eye.

#### Directions for use:

1. Clean hands with a non-moisturizing hand soap.
2. Remove tamper-resistant seal.
3. Apply a generous amount of Clear Clean to the lenses.
4. Clean lenses gently, being careful not to apply too much pressure while rubbing.
5. Repeat steps 3 and 4.
6. Rinse lenses lightly (1 or 2 quick rinses with saline or distilled water).
7. Insert lenses using preferred method



**Note:** If lenses feel dry, dirty or uncomfortable, use Contex Polish Brite, then Clean and soak the lenses in Clear Clean. If the problem persists, professional polishing may be required.

**Safety Packaging:** Use only if tamper resistant seal is intact.

**Warning:** Clear Clean is not for use directly in the eye. To avoid contamination of this product, always close the cap and do not touch tip of container to any surface. If solution changes color or becomes cloudy, do not use. In the case of eye pain, changes in vision, redness or irritation of the eye, consult a physician. Keep this and all medicines out of the reach of children.

### CONTEX POLISH BRITE

- Removes Deposits that can Decrease Comfort & Vision
- Hypoallergenic
- Ensures Optimum Comfort
- Removes Light Scratches

Contex's unique Polish Brite formula cleans and polishes gas-permeable contact lenses. Used weekly, Polish Brite can remove and prevent many types of surface deposits, including hard to remove protein deposits. Polish Brite also removes light scratches. Contex's high quality standards and innovative formulas have provided "clean comfortable" lenses for thousands of patients for over 40 years!

**Indications:** For polishing gas permeable contact lenses prior to disinfecting and cleansing.

#### Directions for use:

1. Clean hands with a non-moisturizing hand soap.
2. Apply a pea-sized amount of Polish Brite to your palm.
3. Add a few drops of water to your palm, mixing the two together.
4. Place the lens in the mixture on your palm.
5. Gently clean the inside, outside, and edges in your palm using your fingers (Be careful not to apply too much pressure while rubbing).
6. Rinse
7. Repeat steps 2 through 6.
8. Rinse lenses completely with water.
9. Clean and Soak the lenses in Contex Clear Clean for at least 6 hours before use.

**Note:** If lenses fog or feel dry, dirty or uncomfortable, clean and soak the lenses in Clear Clean for up to 24 hours. If the problem persists, professional polishing may be necessary. Cosmetic additives or oil-based products cannot be removed except with professional laboratory polishing.

**Warning:** Polish Brite is not to be used directly in the eye! To avoid contamination of this product, do not touch tip of container to any surface. Replace cap after using. If solution dries out or changes color, do not use. If you experience eye pain, changes in vision, redness or irritation of the eye, consult a physician. Keep this and all medicines out of the reach of children.

### PATIENT CLEAR CLEAN & POLISH BRITE ORDER FORM

Product	Size	Price	Qty	Amount
Contex Clear Clean	4oz	\$7.00		
<b>Clear Clean Special:</b> Four(4) 4oz Bottles		\$24.00		
Contex Polish Brite	1oz	\$15.00		
DMV® Ultra GP Contact Lens Remover / Inserter		\$3.00		
<i>(Prices Subject to Change Without Notice) Subtotal</i>				
<b>Shipping and Handling</b> <i>*could be higher in some areas</i>				\$13.00*
<b>Total</b> <i>(\$20.00 minimum order please)</i>				

Name \_\_\_\_\_

Company \_\_\_\_\_

Address \_\_\_\_\_

*Note: We cannot ship to PO Boxes; Delivery in 2-3 weeks.*

City \_\_\_\_\_

State \_\_\_\_\_ ZIP \_\_\_\_\_

Daytime Phone \_\_\_\_\_

Email \_\_\_\_\_

Date \_\_\_\_\_



(circle one)



Card# \_\_\_\_\_

Exp Date \_\_\_\_\_ CCID \_\_\_\_\_

**100% Satisfaction Guaranteed, or Full Refund (less S&H)**

TO ORDER: **Phone 818-788-5836** or **Fax: 818-788-5078**  
-OR-

Mail Order Form w/Check or Money Order Payable to:

**Contex, Inc.**  
**4505 Van Nuys Blvd**  
**PO Box 5750**  
**Sherman Oaks, CA 91413-5750**